
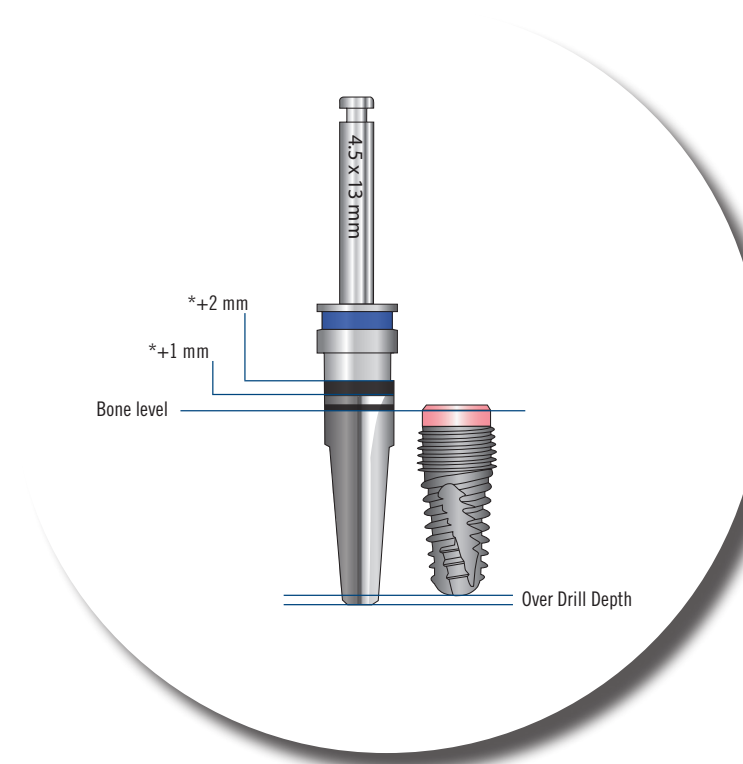


Genesis Drill Sequence and Depth Marking Information for a $\varnothing 4.5 \times 13$ mm implant shown.

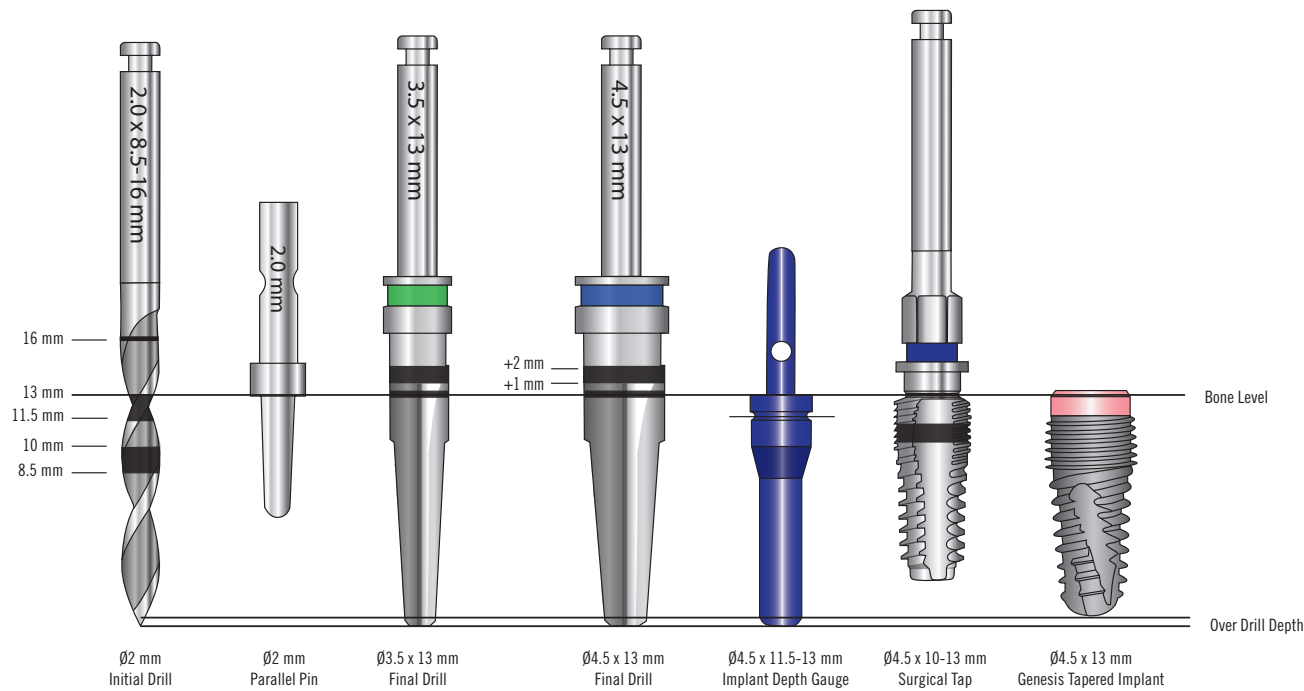

*The Genesis implant drills feature the following laser markings:

- Thin laser mark represents implant length
- Bottom of thick laser mark represents implant length + 1 mm sub-crestal
- Top of thick laser mark represents implant length + 2 mm sub-crestal

The actual drill tip is not included in the designated drill length and must be considered when preparing the osteotomy.

Drill sequence charts for Genesis implant sizes can be found in the Keystone Dental Genesis Implant System Surgical Manual.