

1.8 mm Initial Drill

Twist Step Drill

Parallel Pin/Depth Gauge

1.8 2.2/2.6 2.6/2.9 2.9/3.3 3.4/3.9 3.8/4.1 4.4/4.8 4.6/5.3 5.3/5.7

1 2 3 4 5 6 7 8 9

Drill Extenders

Drill Extender

Ratchet Adapter

Adapter

Parallel Pins

3.5 4.1 5.0 6.0 6.0 Long

Surgical Tap

Taps

Quad Driver

Quad Driver, Latch Type

Drivers

3.5 4.1 5.0/6.0

Quad

Implant Driver

Surgical Ratchet

Spare Instruments

Spares

Implant Ø	Soft Bone	Medium Bone	Hard Bone
SD Ø3.5mm	1+(2)	1+2+(3)	1+2+3+(4)
RD Ø4.1mm	1+2+(3)	1+2+4	1+2+4+(5)
WD Ø5.0mm	1+2+4+5	1+2+4+6	1+2+4+6+(7)
WD Ø6.0mm	1+2+4+5+(8)	1+2+4+5+7+(8)	1+2+4+6+7+8+(9)

(-) Denotes widening of the crestal bone is recommended.