

Drill Sequence Depth Marking Chart

GENESIS ACTIVE™ Drill Sequence and Depth Marking Information for a Ø 4.5 x 13 mm implant shown.

- Thin laser mark represents implant length
- Bottom of thick laser mark represents implant length +1 mm sub-crestal
- Top of thick laser mark represents implant length +2 mm sub-crestal
- The actual drill tip is not included in the designated drill length and must be considered when preparing the osteotomy. Drill sequence charts for GENESIS ACTIVE™ implant sizes can be found in the Keystone Dental GENESIS ACTIVE™ Implant System Surgical Manual.

